

Power Cord for the HT50 Ventilator with Dual Pac Battery

All HT50 ventilators produced since January 2009 include the new Dual Pac Internal Battery System.

An additional change with **this new configuration is the AC power cord which includes an EMC filter box and ferrite**. It is designed to work with the dual battery system to ensure proper electromagnetic interference protection and EMI/RFI shielding.

Here are 3 reminders for using this new power cord.

1. HT50's with Dual Pac can only be used with the new power cord – **DO NOT** use the old version of the power cord.
2. The new power cord, p/n V24-00900-90, is clearly labeled "HT50 Power Supply Cord".
3. To protect the ventilator from being accidentally unplugged, be sure to attach the power cord to the ventilator using the plastic retaining clip.



Contact your Newport Representative for more details.

HTCordwithDualPac Rev. A 0709