









### Filter Usage Guide for Newport e360 & e500 Ventilators

Location	Filter Requirements	
<b>Mainflow Outlet/ TO PATIENT Port</b>	- Disposable Bacteria Filter p/n FLT1801Q /ea (p/n FLT1801Q-C /case) or equivalent, or	
	- Reusable Filter p/n FLT1200P /ea or equivalent	
<b>Exhalation Inlet/ FROM PATIENT Port</b>	<u>With No Filter Heater or Water Collection System:</u>	
	- Disposable Bacteria Filter p/n FLT1801Q /ea (p/n FLT1801Q-C /case) or equivalent	
	<u>With Filter Heater System:</u>	
	- Disposable Filter p/n 3000/03 /ea (p/n 3000/03-C /case) or equivalent, or	
	- Reusable Filter p/n FLT1200P /ea or equivalent	
	<u>With Water Collection System:</u>	
- Disposable Filter p/n 3000/04 /ea or equivalent, or		
- Reusable Filter p/n FTR1800P /ea or equivalent		
<b>Proximal Sensing Port (e500 Only)</b>	Disposable Filter Tubing Assembly Kit p/n FLT1850DC /5pk or equivalent	

**Note:** Always observe directional arrows on filter housings.