

SUBJECT: **New Solution for Water Accumulation in the HT50 Proximal Pressure Tubing and Check Prox Line Alarm**



New HT50 Accessory

Proximal Pressure Adapter Tee
(including cap)

Replacement Cap
for the Proximal Pressure Adapter Tee

Part Number

NP130-22

BE40A




Newport is aware that some of your customers who are using the HT50 Ventilator with a heated humidifier are having problems with frequent Check Prox Line alarms and the frequent need to change the Proximal Filter in the breathing circuit.




The new **Proximal Pressure Adapter Tee** & cap (p/n NP130-22) allows you to tee the proximal pressure sensing tubing in with the breathing circuit tubing, between the ventilator gas outlet and the humidifier inlet (dry side of the humidifier). This eliminates the possibility of water contamination to the proximal tubing and filter.

Caution: The Proximal Filter must be used at all times.

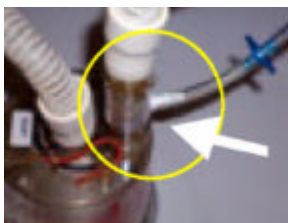

If using a third party humidifier the Adapter Tee can be placed on the inlet port of the humidifier chamber.

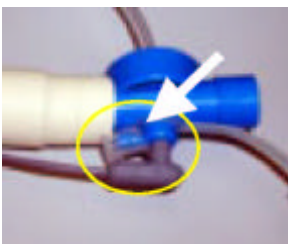

The Replacement Cap that comes with the tee is then used to plug off the proximal port on the HT50 reusable exhalation valve where the proximal tubing would normally connect.

Replacement Caps are sold separately (p/n BE40A).

If you have any questions or need additional information please contact Newport Clinical Education: 1.949.642.3910 ext. 218 or email: clinical@newportnmi.com.